

Record video and capture still images in crystal-clear 4K resolution. The IPS740DS and IPS740DG recorders come with a rich set of features that will upgrade any medical video application. These recorders also integrate with VACS® software for video archiving and collaboration.

Simultaneously record files to the internal hard drive and external USB drive. 4 TB of internal storage allows for extended recording times. Also record and playback video content in 3D formats.

Use the integrated 5 inch touchscreen, or switch to an external monitor to control all recording, playback, saving functions, or to preview the active video signal.



### Focus on Compatibility

IPS740DS and IPS740DG connect with medical image generating devices, and with patient information systems, to accurately record and store video or captured still images.

Input signal options:

- 3G-SDI (IPS740DS)
- 3G/12G-SDI (IPS740DG)
- DisplayPort 1.2
- HDMI 2.0

File management options:

- DICOM & HL7 Integration
- DICOM Modality Worklist
- DICOM MPPS
- DICOM C-Store
- HL7 Appointment Booking

Additional medical A/V integration options

- Patient search & query
- Medical printer compatible
- Gigabit Ethernet
- 4 x USB 3.0 ports
- Audio inputs/output
- 2 x foot pedal inputs



**CAPTURE** button. Capture can also be initiated by using an attached monitor (touchscreen or mouse), or with VACS software.

**RECORD** button. Video recording can also be initiated by using an attached monitor (touchscreen or mouse), or with VACS software.




Select the **MONITOR** icon to switch to the connected external monitor. When in this mode, touchscreen can also be used as a touchpad.

Create a new **TASK** when plus (+) symbol is shown, or close an active task when minus (-) symbol is shown.

- Return to **HOME** screen icon.
- FILE** icon manages tasks, including: search, modify, copy, delete, DICOM store, and print.
- WORKLIST** icon retrieves patient information that is stored on the worklist server.
- SETUP** icon opens settings for input sources, file types, DICOM, server, network, local time, foot pedal, printing layout, and system options.



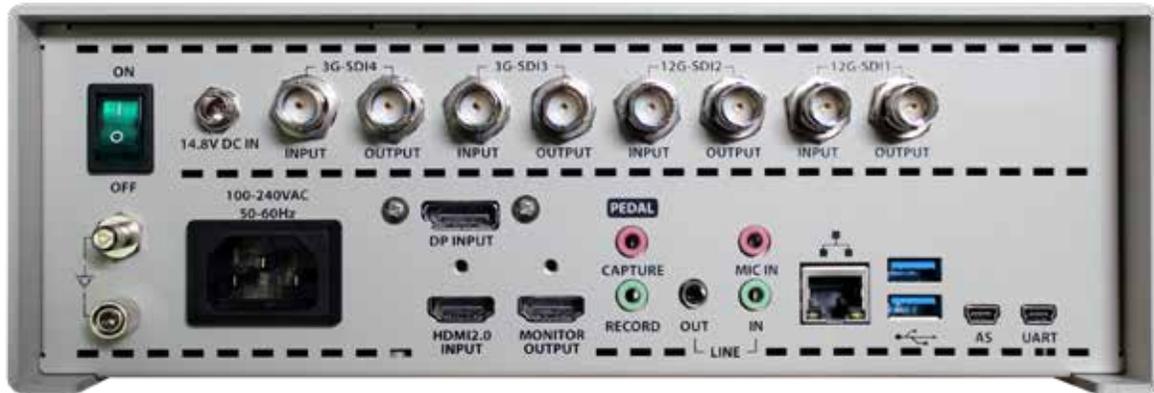

IPS740DG with 3G and 12G-SDI

### General Features

Item	Description
Main Functions	Video recording Up to 3840 x 2160P 60Hz 4096 x 2160P 60Hz
	Still image capture Up to 3840 x 2160P 4096 x 2160P
	Standalone preview Front LCD screen
	Menu access Front LCD screen (5 inch touch enabled) GUI on HDMI monitor output.
	DICOM, HL7 DICOM Store (Image) Modality Worklist MPPS Appointment Booking (HL7)
DICOM store Direct Access to PACS	
HDMI 2.0	
DisplayPort 1.4	
Inputs	SDI IPS740DS: 3G-SDI IPS740DG: 12G-SDI, 2SI, Quad
	Audio 3.5mm Stereo x 2ea Mic in, Line in
Outputs	Monitor output HDMI 2.0
	Loop-through IPS740DS: 3G-SDI IPS740DG: 12G-SDI, 2SI, Quad
	Audio 3.5mm Stereo line out



IPS740DS with 3G-SDI

Item	Description
Data I/O	USB 3.0 2 on front, 2 on back
	RS-232C Mini-USB
	Ethernet RJ45 x 1 (10/100/1000M)
	Keyboard / Mouse USB type
Foot pedal	3.5mm stereo jack x 2 Recording, Capture
	Recording resolution 4096x2160p60 3840x2160p60 1920x1080p60
Recording and Capture	Encoding formats H.264, H.265
	Recording file format MP4
	Capture image format BMP, JPEG, DICOM
Audio encoding format AAC	3D video formats 4K line alternative (SBSH conversion) 4K side-by-side half
	Internal hard drive 4TB
Storage	External USB storage USB flash drive USB HDD
	USB file system FAT32, NTFS
	Network file system CIFS
	Network transfer FTP (client)
Power requirements	AC 100-240V / 50-60Hz, 1.0A - 0.6A (max)
	Power consumption 60W (max)
Dimensions	260mm (10.2 in.) wide x 95mm (3.7 in.) high x 330mm (13 in.) deep
	Weight IPS740DS: 4.0kg (8.8lbs.) IPS740DG: 4.1kg (9.0lbs.)
Temperature	Operating: 0° to +40° C (+32° to +104° F) Storage: -20° to + 60° C (-4° to +140° F)
	Humidity Operating: 10 - 85% RH Storage: 10 - 85% RH
Compliance & Certifications	CE-MDR 2017/745 Class 1 Medical Device, UL 60601-1, IEC60601-1, EN 60601-1-2, FCC Part 15B, CCC, ISO9001, ISO13485, RoHS

Specifications are subject to change with or without notice. Inspire

[www.inspiremeditech.com](http://www.inspiremeditech.com)

## INSPIRE MEDITECH PRIVATE LIMITED

407 Red Rose Building 49-50, Nehru Place New Delhi-110019

Office +91 011 47065575

H/P+919811078156,919313899647,919717480657

sales@inspiremeditech.com | www.inspiremeditech.com

Foreseeson GmbH  
Industriestrasse 38a  
63150 Heusenstamm, Germany  
Tel: +49 6104 64398 0  
Fax: +49 6104 64398 11

Foreseeson Korea

Foreseeson UK Ltd.  
1 Wolsey Road  
East Molesey  
Surrey  
KT8 9EL  
United Kingdom  
Tel: +44 (0) 208 546 1047

Foreseeson (Shanghai)  
Medical Equipment Co., Ltd.  
Room 307, 3F  
No. 56, 461 Hongcao Road  
Caohejing Development District  
Xuhui, Shanghai 200233  
Tel: 86-21-6113-4188